

Encrypted Disc Instructions


Encrypted CDs only run on Windows PCs (not on Apple devices).

1. Insert the disc into computer CD/DVD drive.


2. Select "Run dcstart.exe".



3. Select "View Images" (this may take a few seconds).

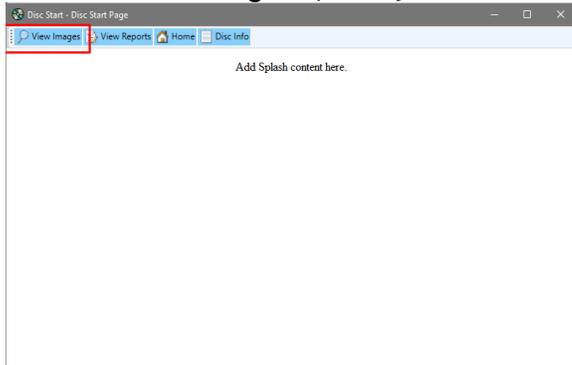

4. Select "I Agree".

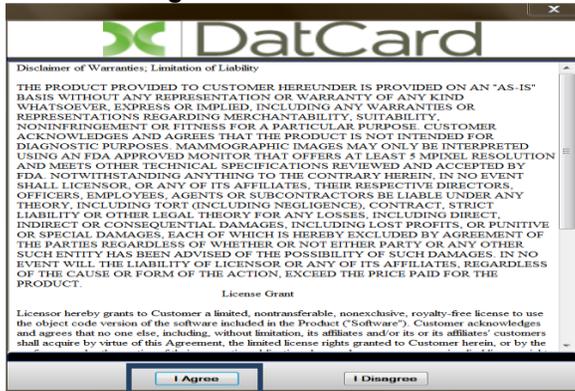


5. Type in the disc password (access code) that you received from the Imaging Facility or Hospital. The password is case sensitive.



You will get 2 attempts at entering the password. If you are rejected because of typos, eject the CD/DVD and start again at step #1.

6. Select an exam from the list and click the "Display" button (lighter gray color indicates the exam is selected).

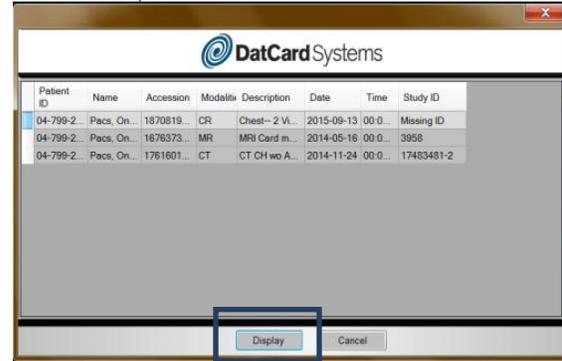


7. Top left of toolbar: Left diagram takes you back to list of exam(s) / images.




Right diagram takes you to the exam report(s) (if available).

8. Select each tab to view the exam results (report).

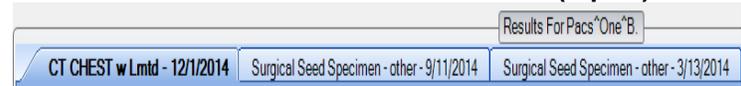

9. To print a report: Right click, select Print; or, select CTRL + P on your keyboard and select Print.

10. Close Program: Select the X in the upper right corner of the application.

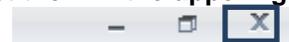

MEDICAL INSTITUTIONS EXPORTING DICOM IMAGES TO PACS: We recommend you contact your local IT specialist (or PACS administrator) for assistance.

OPTION I (sending directly to AE Title)

- Follow steps 1-8 above
- Select The Export Tab
- Select DICOM Send
- Enter your Destination IP Address
- Enter your Destination AT Title
- Enter your Destination Port
- Enter your Calling AE Title
- Select Configure; Select Send

OPTION II (exporting to a local folder)

- Follow steps 1-8 above
- Select Study Export
- Enter your Export Path
- Select Export
- You will then need to retrieve the study from your local PC folder and send to your configured destination.